


BD Eclipse™ Needle


For Luer Lock Syringes Only

QUICK REFERENCE CARD

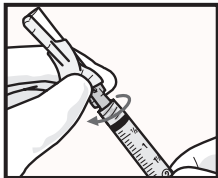


1a. Attachment

Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.

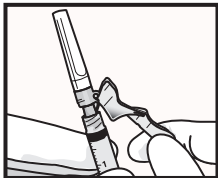


1b. Attachment

Pull back on the safety cover toward the syringe and away from the needle.

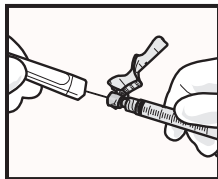

1c. Attachment

Grasp the syringe with one hand and with the other pull the clear needle shield straight off.

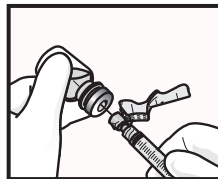

2. Aspiration


Draw up medication in accordance with established protocol.

NOTE: If changing needles, activate safety cover prior to removing the needle from the syringe.

BD Eclipse™ Needle


For Luer Lock Syringes Only

QUICK REFERENCE CARD

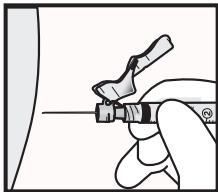

3. Injection

Administer medication in accordance with established protocol. For user convenience, the needle “bevel-up” position is oriented to the safety cover.

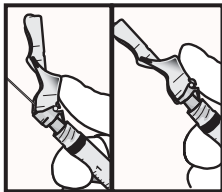

4. Activation

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward over the needle until you hear and/or feel it lock.

Visually confirm that the needle tip is covered. After activation, immediately discard into an approved sharps container. Do not reuse.

Things to Remember:

- For Luer Slip Syringes, use BD Eclipse™ Needles with SmartSlip™ Technology.