



Helping all people live healthy lives




# BD Eclipse™ Needle

## For Luer Lock Syringes Only

# IMPORTANT USAGE GUIDELINES

### 1. ATTACHMENT

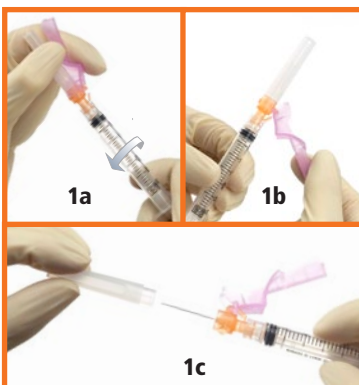

1a. Firmly attach needle onto a Luer Lock syringe with a push and clockwise twist.  
 1b. Pull back on the safety cover toward the syringe and away from the needle.  
 1c. Grasp the syringe with one hand and with the other pull the clear needle shield straight off.

### 2. ASPIRATION




Draw up medication in accordance with established protocol.  
**NOTE:** If changing needles, activate safety cover prior to removing the needle from the syringe.

### 3. INJECTION



Administer medication in accordance with established protocol. For user convenience, the needle “bevel up” position is oriented to the safety cover.

### 4. ACTIVATION

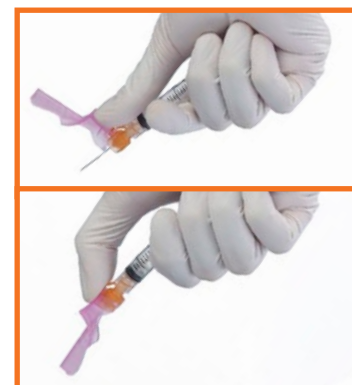

After injection, immediately activate safety cover as follows:

- Using the same hand holding the syringe, center thumb or forefinger on the finger pad area of the safety cover.
- Push the cover forward over the needle until you hear and/or feel it lock.

Visually confirm that the needle tip is covered. After activation, immediately discard into an approved sharps collector.

### THINGS TO REMEMBER

- For Luer Slip Syringes, use BD Eclipse™ Needles with SmartSlip™ Technology.
- Aseptic technique and proper skin preparation of the site are essential.
- Discard BD Eclipse™ Needle and syringe in an approved sharps collector after single use. Follow your institution's policy for safe disposal of all medical waste.
- Do not reuse.

**BD Medical**  
 1 Becton Drive  
 Franklin Lakes, NJ 07417  
[www.bd.com](http://www.bd.com)