


BD Integra™ 3ml Syringe



with Retracting BD PrecisionGlide™ Needle

QUICK REFERENCE CARD

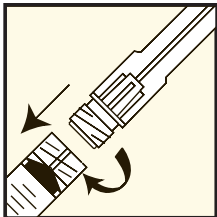

1. Attachment


Attach the Retracting BD PrecisionGlide™ needle onto the BD Integra™ syringe with a push and a clockwise twist. Pull the shield straight off of the needle.

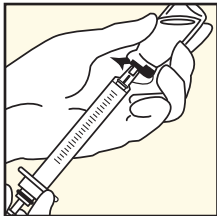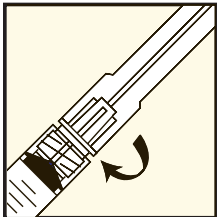

2. Aspiration

Draw up medication in accordance with established protocol.

Note: If pre-attached, tighten the needle to the syringe prior to use.

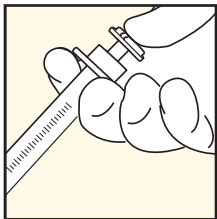


3. Injection

Administer medication in accordance with established protocol.

BD Integra™ 3mL Syringe

with Retracting BD PrecisionGlide™ Needle

QUICK REFERENCE CARD

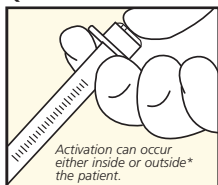

4. Activation


Push plunger rod to activate the safety mechanism. Depress until you sense two "clicks."

When activating device inside the patient, do not advance needle while depressing plunger rod.



- * Activation of the device may cause minimal splatter. When activating the device outside of the patient, hold the device downward at arm's length, and activate while pointing away from self and others. Keep device within sight at all times to avoid accidental needlestick injury.

Things to Remember:

- BD Integra™ PrecisionGlide™ needle and BD Integra™ syringe are not a standard luer connection.
- After pressurizing the medication vial, do not continue to press the plunger rod past the point of resistance, as this may cause premature activation.
- Do not use this product to draw blood or for venipuncture. Do not reuse.

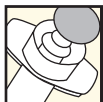

Before

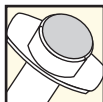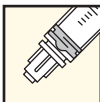

After

5. Retraction

The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

There is no exposure to the used needle. Immediately discard into an approved sharps container.