



BD Integra™ 3mL Syringe

with **RETRACTING**
BD PrecisionGlide™ Needle

IMPORTANT USAGE GUIDELINES

1. ATTACHMENT



Attach the BD Integra™ needle onto the BD Integra™ syringe with a push and a clockwise twist. If pre-attached, tighten the needle to the syringe prior to use.

2. ASPIRATION



Pull the shield directly off of the needle. Draw up medication in accordance with established protocol.

3. INJECTION



Administer medication in accordance with established protocol, making sure that all medication has been dispensed.

4. ACTIVATION



Push plunger rod to activate the safety mechanism. Depress until you sense two "clicks".

Activation can occur before or after† needle is withdrawn from the patient.

5. NEEDLE RETRACTION



The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

Discard immediately in the nearest sharps container.

ISO NEEDLE HUB/ PACKAGING CHART

25G

23G

22G


21G

BD INTEGRA BLUNT FILL AND FILTER NEEDLE²

18G Fill

18G Filter

THINGS TO REMEMBER:

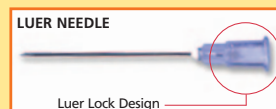

- BD Integra™ Needle and Syringe **DO NOT** HAVE A STANDARD LUER CONNECTION. Conventional Luer Lock needles cannot be used with the BD Integra™ Syringe.
- The BD Integra™ Syringe is not compatible with valves or other needleless systems.
- When pressurizing a medication vial, do not continue to press the plunger rod past the point of resistance, as this may activate the device prematurely.
- If Activation is performed before needle is withdrawn from patient, do not advance needle while depressing plunger rod.
- † If Activation is performed after needle is withdrawn from patient, minimal splatter may occur. When activating the device outside of the patient, hold the device downward at arm's length, and activate while pointing away from self and others. Keep device within sight at all times to avoid accidental needlestick injury.
- **DO NOT** use this product to draw blood or for venipuncture.
- When delivering viscous medications (Ativan, Penicillin G) use a 22G BD Integra™ Retracting Needle or larger, and inject at a controlled rate.
- **DO NOT** reuse.

1 Colors reflect worldwide ISO color standards to enhance worldwide sourcing and product supply.

2 Not for injection